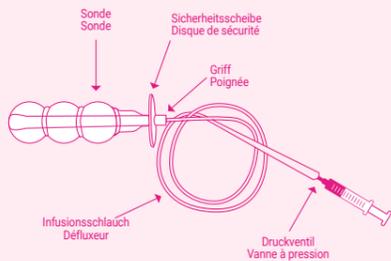
Fig./ Abb.1

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EN ELLEN REF A.1005G

Vaginal probe for contraction training

Pathologies of the pelvic floor can be associated to a hypertonic or hypotonic condition of the pelvic muscles.
ELLEN is a powerful partner in the fight against invalidating discomforts in everyday life and in the prevention of childbirth damages.

COME SI USA:

Before the use of **ELLEN**:
• Remove all the air contained in the probe by connecting the syringe to the valve and drawing the air.
• Perform a warm 5-minute hip bath (fill a bucket with warm water and cleanse the vaginal area with Sapigen K), in order to completely relax the perineal muscles.

The recommendation is to insert the probe while in supine position, with legs slightly spread (Fig.1).

Fill the syringe with 80/90 ml of air or warm water, the temperature should be between 40° and 45°C (warm tap water). The temperature can be controlled with a thermometer.

Thoroughly lubricate the probe with a lubricating gel and delicately insert it fully into the vagina, until the safety disc is in contact with the vulvar commissure. Relax the legs to block the safety disc with the inner thigh.

Connect the syringe filled with air or water to the probe's pressure valve (Fig.2). The use varies according to the pathology which needs to be treated (see the following **Treatment Scheme**).

HYPOTONIC CONDITION

- Associated pathologies:**
- Urinal incontinence
 - Faecal incontinence or flatulence
 - Uro-genital prolapse
 - Pelvic heaviness
 - Dyspareunia and sexual hyposensitivity

INDICATIONS

The pubococcygeus muscle takes part both in the regulation of the urinary flow and in the control of the birth-giving apparatus.
Some debilitating factors might be pregnancy, obesity and menopause.
Urinary incontinence does not necessarily need pharmaceutical treatment. From time to time simply contracting and relaxing the voluntary pubococcygeus muscle for few seconds can help strengthen the pelvic floor muscles and perineal area. **ELLEN** is indicated for both preventing and controlling the pathology.

TREATMENT SCHEME

Insert the deflated probe in the vagina and connect the syringe loaded with air or water to the pressure valve, inflate the probe until it creates a feeling of pressure (80/90 ml).
Squeeze and release **ELLEN** through internal contraction and relaxation movements, without contracting glutes or abdominal muscles (Fig.3).

Prevention: train by performing 10 slow tonic contractions of the muscles alternated with 10 quick phasic contractions. Tonic contraction must be performed for 5 seconds and then rest for 10 seconds in between (Fig.4); phasic contractions must be performed for 2 seconds and then rest for 4 seconds in between (Fig.5). At the end of each sequence rest for 30 seconds. Repeat the whole sequence 3 times for a total of 60 contractions (30 tonic + 30 phasic), twice a week.

Treatment: train by performing 10 tonic contractions (slow) of the muscles + 10 phasic contractions (quick) + 10 strength-increasing contractions (gradually increasing the strength). Tonic contraction must be performed for 5 seconds and then rest for 10 seconds in between; phasic contractions must be performed for 2 seconds and then rest for 4 seconds in between; perform strength-increasing contractions by gradually increasing the strength and then rest for 10 seconds in between (Fig.6). At the end of each sequence rest for 30 seconds. During the first two weeks, train daily. During the first week, start with 30 contractions per day (10 slow, 10 quick, 10 increasing strength) and increase to 60 contractions during the second week. From the third week, repeat the whole sequence 3-4 times for a total of 90-120 contractions (30-40 tonic + 30-40 phasic + 30-40 strength-increasing) and decrease to 3 or 4 training days per week.

Maintaining: perform the sequence indicated in "Treatment", 1 or 2 times a week for a 2-month period.

After each training session, deflate the probe by drawing the air/water back into the syringe through the pressure valve. Hold the handle and extract the **ELLEN** device very slowly.

SIDE EFFECT
Overtraining should be avoided as it could lead to muscular fatigue and increase incontinence as an undesirable side effect.

At the end of the maintaining period dispose of the probe even if intact.

HYPERTONE

- Associated pathologies:**
- Pelvic pain
 - Vulvar-vaginal atrophy
 - Vulvar Lichen
 - Post-surgery or post-partum vulvar-vaginal scars
 - Vestibulitis- vulvodynia

INDICATIONS

A hypertonic pelvic floor is a condition in which the muscles stiffen and contract, losing their elasticity and mobility. This leads to pain, fatigue, dyspareunia and urinary or fecal deficits. During pregnancy, it is recommended to prevent laceration damages caused by childbirth and therefore, in the third trimester of pregnancy, it is very useful to train the pelvic muscles, giving them increased elasticity. Thanks to **ELLEN's** softness and elasticity it's possible to create tailored therapeutic treatments aimed at achieving the desired results: better tissue elasticity, ready for maximum dilation during delivery.

TREATMENT SCHEME

Insert the deflated probe in the vagina.

Treatment: connect the syringe loaded with air or water to the pressure valve, inflate the probe (as per professional indication), until it creates a feeling of pressure.

After the first 5 minutes, further inflate with air/water until the feeling of pressure strengthens slightly and hold in position for 5 more minutes. Repeat this procedure during a 20 minute therapy session. After the 20 minute session, it is advised to extract the device while it is inflated (or very slightly deflated), so to stretch the vaginal duct.

ELLEN can also be used by patients undergoing a male to female gender transition in the post-operative period, to avoid the neovaginal stenosis.

Maintaining: perform the sequence indicated in "Treatment", 1 or 2 times a week for a 2-month period.

SIDE EFFECTS

During the treatment, perianal pain could arise. This symptom is temporary and will disappear after the **ELLEN** session.
In case of severe vaginism, the pain could increase during the first weeks of treatment, especially during the first few days. In this case, it is not recommended to stop the therapy but to use a mild painkiller that will reduce pain and allow to the **ELLEN** session to continue
If the pain is very intense, it is recommended to stop the **ELLEN** session and contact a professional for further advice.

At the end of the maintaining period dispose of the probe even if intact.

ELLEN CLEANING AND CARE

Wash the **ELLEN** device under a warm water stream and clean the probe with soap. Dry with a cloth and store **ELLEN** in a dry place.

WARNING

- For a correct use of the **ELLEN** device or for any information, please refer to your family doctor or physician.
- **The maximum water capacity to inflate the device is 150ml. A higher volume could damage the probe.**
- **ELLEN** is a medical device and should therefore be used as a physical treatment for the above-mentioned pathologies.

SPECIFIC WARNINGS

- It is strictly not recommended to use **ELLEN** for any other conditions.
- Keep away from children.
- Do not freeze.
- Do not boil.
- Do not heat in a microwave oven.
- Do not cut.
- Do not swallow.
- Dispose of the device carefully after use.
- Only wash with neutral detergents.
- The use of **ELLEN** is strictly personal.

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